

ambre



Ambre 9000
Sit Stand Chair

DESCRIPTION

The Ambre 9000 has a stable, welded frame with 360° castors and a safe and easy to use brake. The frame is small yet still offers plenty of room for the feet. As a standard, this chair is longer at the back than at the front to keep the risk of tipping over to a minimum. The brake system, with its individual brake legs and durable brake pads, ensures maximum friction on all types of surfaces. The brake has a computer calculated lift curve so it can be applied using minimal force. The ground clearance has been set to prevent snagging on carpets and thresholds. The brake bracket is adjustable in stages, allowing the armrests to be adjusted in width whilst still ensuring the brakes are easy to apply.

FEATURES



- Backrest
- Adjustable armrests
- Electric or manual: rise and descend
- Lifting range of 20 cm (or 14 cm). Electric 28 cm (or 20 cm).
- Frame
- Brake lever
- Footrest available
- 10cm Castors

MOVING AROUND

HORIZONTALLY

Moving around while seated in the Ambre 9000 is easy. The castors turns smoothly in different directions, and the well designed frame provides plenty of legroom. Angling the seat and back forwards provides a firm base for the feet. The castors are available in a range of materials and sizes to allow the chair to be moved across thresholds and different kinds of surfaces.

VERTICALLY

There is approximately 20 cm difference between the average height of a dining table and the height of a kitchen sink. To achieve a good working position at both, it is important for the chair to be sufficiently height adjustable. The Ambre 9000 has gas spring controls with a lifting range of 20 cm.

VERTICALLY - Electric

The Ambre 9000 has an electric height adjustment with a range of 28cm. This substantially increases the users range whilst seated in the chair.

NOTE: The brakes must always be applied before using any of the functions on the chair including the height adjustment. this is important to minimize any risk of injury!

MOVING SIDEWAYS AND OUT OF THE CHAIR

The Ambre 9000 makes it easy for the user to move sideways out of the chair for example if you need to transfer into a bed, wheelchair or a toilet. Place the chair next to you, then apply the brake. Adjust the seat to the right height, lower the armrest and move across.

Folding armrests are available as an optional extra.



SPECIFICATIONS



Ambre 9000 Sit Stand Chair (manual)		Ambre 9000 Sit Stand Chair (electric)	
Frame	48cm x 53cm	Frame	48cm x 53cm
Seat angle	+15° / -8°	Seat angle	+15° / -8°
Low back	Adjustment of height, angle and depth.	Low back	Electric adjustment. For electric chair
High back	Adjustment of height, angle and depth.	High back	Electric adjustment. For electric chair
Castors	10cm castors for all types surfaces	Castors	10cm castors for all types surfaces
Seat height & rotation	45cm - 65cm 360°	Seat height & rotation	47cm - 75cm 360°
Seat	44cm x 48cm	Seat	44cm x 48cm
Attached back	Low 38cm x 26cm, high 40cm x 34cm		Low 38cm x 26cm, high 40cm x 34cm
Armrest	35cm	Armrest	35cm
User weight	55 - 90 kg (Specify when ordering)	User weight	55 - 90 kg (Specify when ordering)

DESIGN

Seat Plate
In a range of sizes and materials.



Seat Angle
(+15° to -8°)
Gas, crank or electric.



Back Plate & Mechanism
Select required design.



Seat Height
Electric or manual



Controls
Select design & position.



Frame
Range of variants.




Castors
For different kinds of flooring and in a range of sizes.



Upholstery
Available in plush, woven fabric & skai



Accessories
Freedom of choice with a huge range.

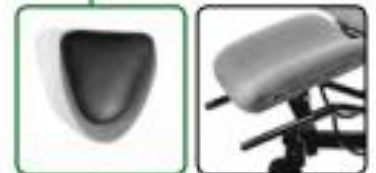

=



Contact Us:

Phone: 01522 420091

Email: enquiries@ambrestone.co.uk

Website: www.ambrestone.co.uk



ENRICHING LIVES THROUGH MAXIMUM POSTURAL
SUPPORT AND PRESSURE MANAGEMENT

Ambre-stone Ltd
Fosters Yard
Station Road
Langworth
Lincoln
LN3 5BD

01522 420091
enquiries@ambrestone.co.uk
www.ambrestone.co.uk