


T Thinking

PRATICO
inserito®
SEAT


SEAT

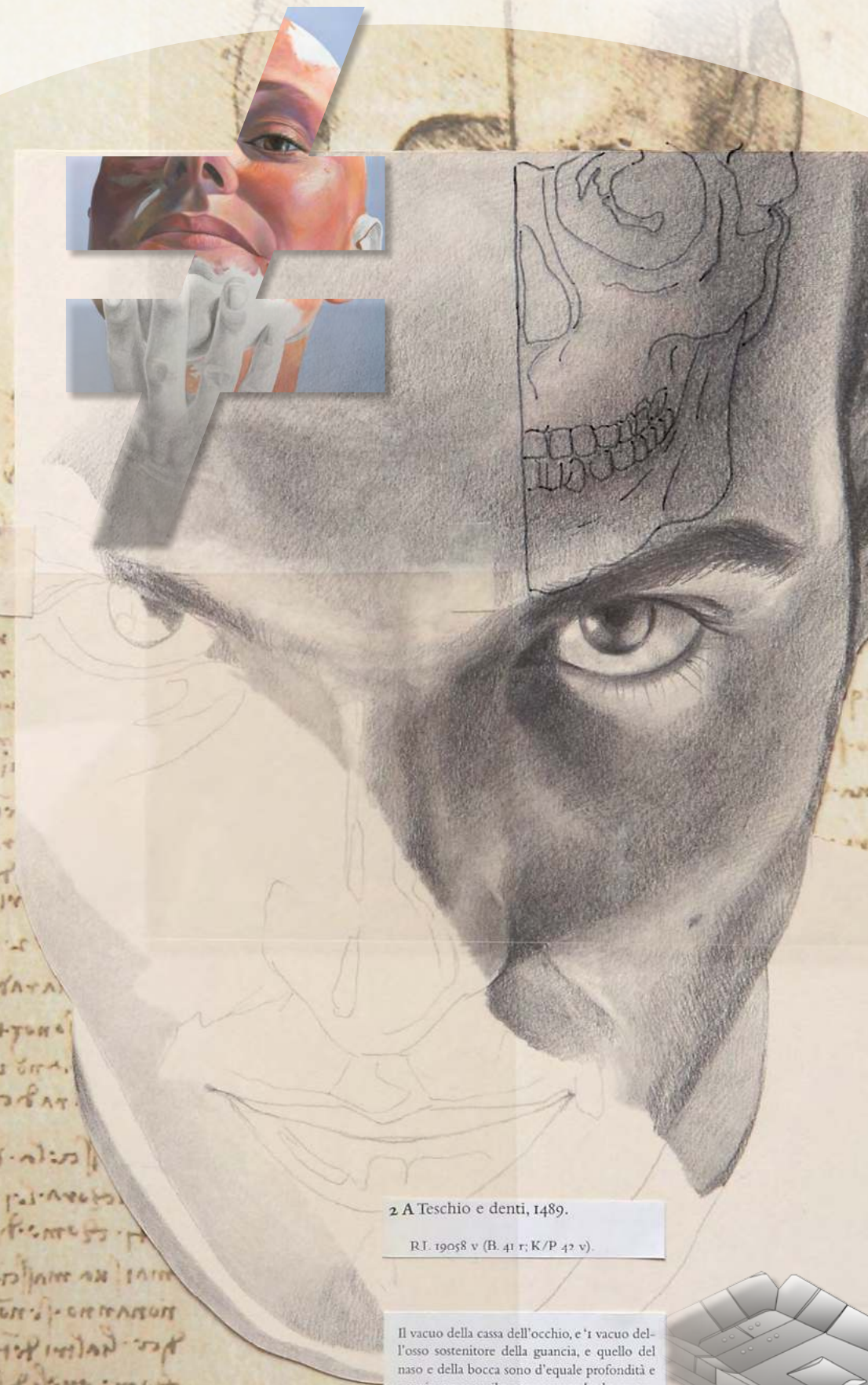

2 A Teschio e denti, 1489.

R.I. 19058 v (B. 41 r; K/P. 42 v).

Il vacuo della cassa dell'occhio, e 'i vacuo dell'osso sostenitore della guancia, e quello del naso e della bocca sono d'eguale profondità e terminano sotto il senso comune [sede centrale delle sensazioni] per linea perpendicolare. E ciascuna d'esse vacuità ha tanto di profondità quant'è la terza parte del volto dell'omo, cioè dal mento a capegli.

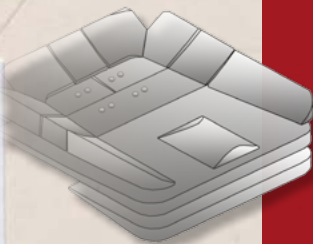

A NEW SOLUTION
INDIVIDUALLY CUSTOMISABLE

A NEW SOLUTION INDIVIDUALLY CUSTOMISABLE Construction kit of the individually customisable seat Perfect reproduction of anatomical shapes



Why Pratico ?

- ENHANCED PROPELLING
- LIGHTWEIGHT
- EASY TO HANDLE AND LOW PROFILE



**STRUCTURAL KIT MODULAR
AND CONFIGURABLE
WITH DIFFERENT DENSITY**



**Free Shapes
PADDING KIT**



**STABILIZING
AND EASY TO USE**

CLINICAL INFORMATION / DESTINATION OF USE

The pelvis positioning system Inserto Seat Pratico is recommended preferably, but not exhaustively, for users with mobility limits, presenting mild deformities and slight need of postural containment and balance, in particular:

- Those who need a mild positioning support;
- Those who lead an active life and need a cushion easy to transfer from and transport;
- Those who need good support and lateral containment of the legs, with a slight reduction in the risk of pressure sores and / or using orthoses to control deformities;
- Those with mild leg length discrepancies, wind swept, pelvic rotations, posterior and anterior pelvis tilt;
- Those who need lateral legs control to achieve better self trunk control.

The pelvis positioning system has to be used with any supporting/wheelchair base properly fixed through suitable gripping systems.

PTS Pelvic Total Support

TECHNICAL FEATURES

Each positioning system Inserto Pratico includes:

- Structural kit with different density, composed by a flat base of construction and 9 closed cells polyethylene inserts, customisable and configurable useful to individually support or correct the posture of the pelvis during the life time use.
- Padding kit, washable with a gentle detergent, which is composed by a layer memory foam with low spring back action in the superior part and a layer of open cells polyurethane foam in the posterior part.
- New 3D cover, air-exchange, incontinent, not flammable, antibacterial. It can be washed in tepid water with a light detergent. Do not tumble dry. The fabric doesn't require the use of disinfectants.

SPARE PARTS ■ Flat base of construction ■ Cover ■ Kit of padding

WARNINGS


The Positioning System Inserto Seat Pratico, in its design integrity and numerical and dimensional completeness of the supplied components (structural kit, padding kit, cover) can be easily adapted to sizes/ morphology/deformities of the user. This kind of operations make the commissioning referable to a serial device.

Alternatively

The commissioning of the Positioning System Inserto Seat Pratico, deprived or modified, even partially, of its design integrity of numerical and dimensional completeness of the supplied components, built as per written prescription of a professional operator in function of the anatomy/morphology/deformities of the user, through the measurement detection and direct trials, can be referable to a custom device.

For partially sighted users this document can be visualised in PDF format (with the possibility to enlarge the text) on the web site www.versainserto.com



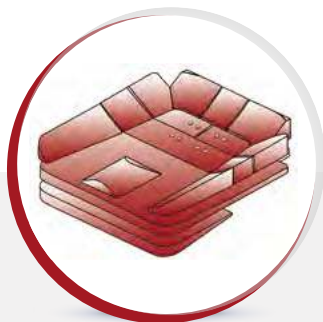

Inserto Pratico

12/13 MODULAR CONFIGURABLE INSERTS

"NEW 3D" COVER
STRECH, INCONTINENT, AIR-EXCHANGE.

° Each postural kit is ordered based upon the size of mobility/wheelchair device.


INSERTO PRATICO SIZES TABLE	PAEDIATRIC						ADULT									SIZES (cm.)	
	2025	2525	2530	3030	3038	3638	3642	4040	4045	4245	4250	4545	4550	4845	4850		
MODEL	2025	2525	2530	3030	3038	3638	3642	4040	4045	4245	4250	4545	4550	4845	4850		
Effective width (cm)	20	25	25	30	30	36	36	40	40	42	42	45	45	48	48		
Max width achieved with adaptation (cm)	22	27	27	32	32	38	38	42	42	44	44	47	47	50	50		
Min. width achieved with removal of material (cm)	18	23	23	28	28	34	34	38	38	40	40	43	43	46	46		
Effective length (cm)	25	25	30	30	38	38	42	40	45	45	50	45	50	45	50		
Min. length achieved with adjustment through seat notches and removal inserts (cm)*	19	19	24	24	32	32	36	34	39	39	44	39	44	39	44		
Min. and max. range of anterior height of flat base of construction 5 (cm)**	2,5/3		** height referred to models 2025 and 4850														
Min. and max. range of posterior height of flat base of construction + buttocks inserts 7 (cm)**	5,5/7,5		** height referred to models 2025 and 4850														
Weight of positioning system (min/max): 0,6 kg/2,2 kg																	

* possible and further reductions due to removal of material

° Max load (referred to model 4850): 135 kg

CONVERSION TABLE

INCHES	8	10	12	12,5	13	13,5	14	15	15,5	16	16	16,5	16,5	17	17,5	17,5	17,5	18	18,5	18,5	19	19,5	19,5	20	20	22
CM.	20	25	30	32	33	34	36	38	39	40	40,5	42	42,5	43	44	44,5	45	46	47	47,5	48	50	50,5	52	52,5	55

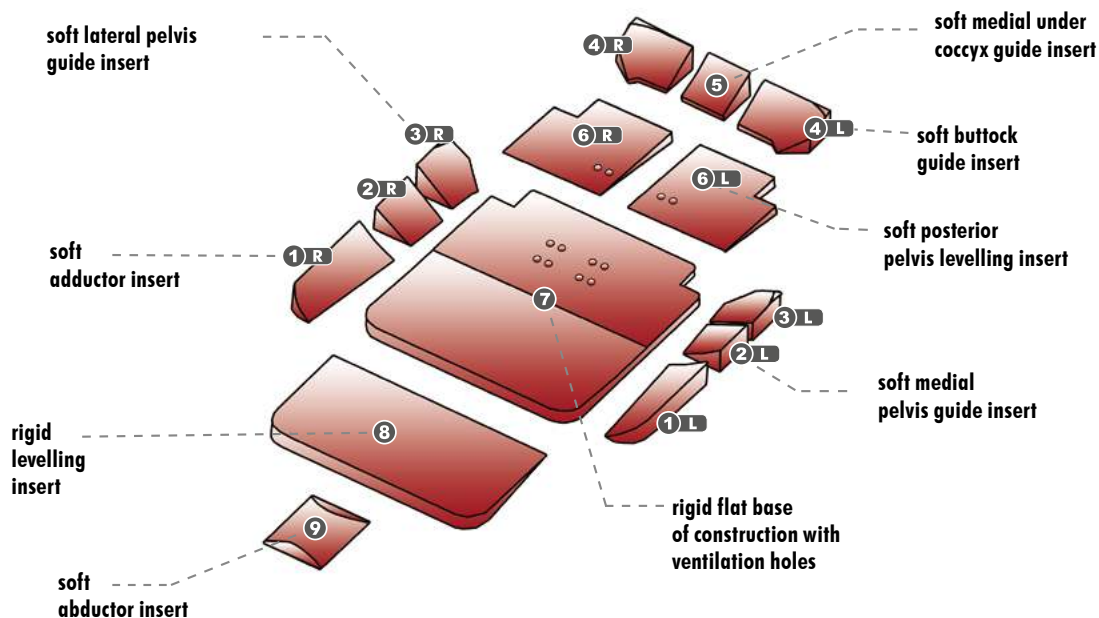



Customisations of Pratico



SEPARATE ISCHIAL-COCCYX
POSTERIOR SUPPORT



ADJUSTMENT OF TROCHANTERIC
SUPPORT PREVENTING ISCHIAL LOADS



ANTI-THRUST OF THE PELVIS



ADDUCTION FOR EASY
PROPULSION



PELVIC OBLIQUITY



ANTERIOR / POSTERIOR
PELVIS TILT



CLOSING ANGLE TRUNK/PELVIS

INDIVIDUALLY
CUSTOMISABLE

Construction kit of the
individually customisable seat



AN INNOVATIVE SOLUTION

A range developed to answer the individual postural needs (as per prescription) and numerous pathologies throughout the whole life cycle of use. In order to maintain its features it is necessary to periodically check the conservation and wear of all the parts composing the device as well as all hook and loops stripes.

www.versainsero.com

1 ADAPTABILITY MAN/WOMAN

Finally a solution that makes the difference with a fully personalised fit.

2 ADAPTABILITY TO THE INDIVIDUAL MORFOLOGY AND ANATOMY

Unlimited possibilities to realize the customised postural solution.

3 PTS PELVIC TOTAL SUPPORT

A perfect fit to anatomy of the pelvis and related body segments, by encouraging an active response of the musculo-skeletal system. The distribution of the loads provided by the PTS reduces the mechanical stress on the deep tissues and significantly enhances the prevention of decubitus ulcers.

4 FREE SHAPES PADDING KIT

The innovative modular padding ensures maximum comfort and reliable protection.






MANUFACTURED BY:



Pro Medicare srl
Via Montagna Z.I. - Lotto 41
72023 Mesagne (BR) - ITALY
Tel. +(39) 0831 777840
Fax. +(39) 0831 730739
Email: sales@promedicare.it
www.versainserto.com
www.promedicare.eu

CERTIFIED COMPANY WITH QUALITY
MANAGEMENT SYSTEM ISO 9001 / EN ISO 13485

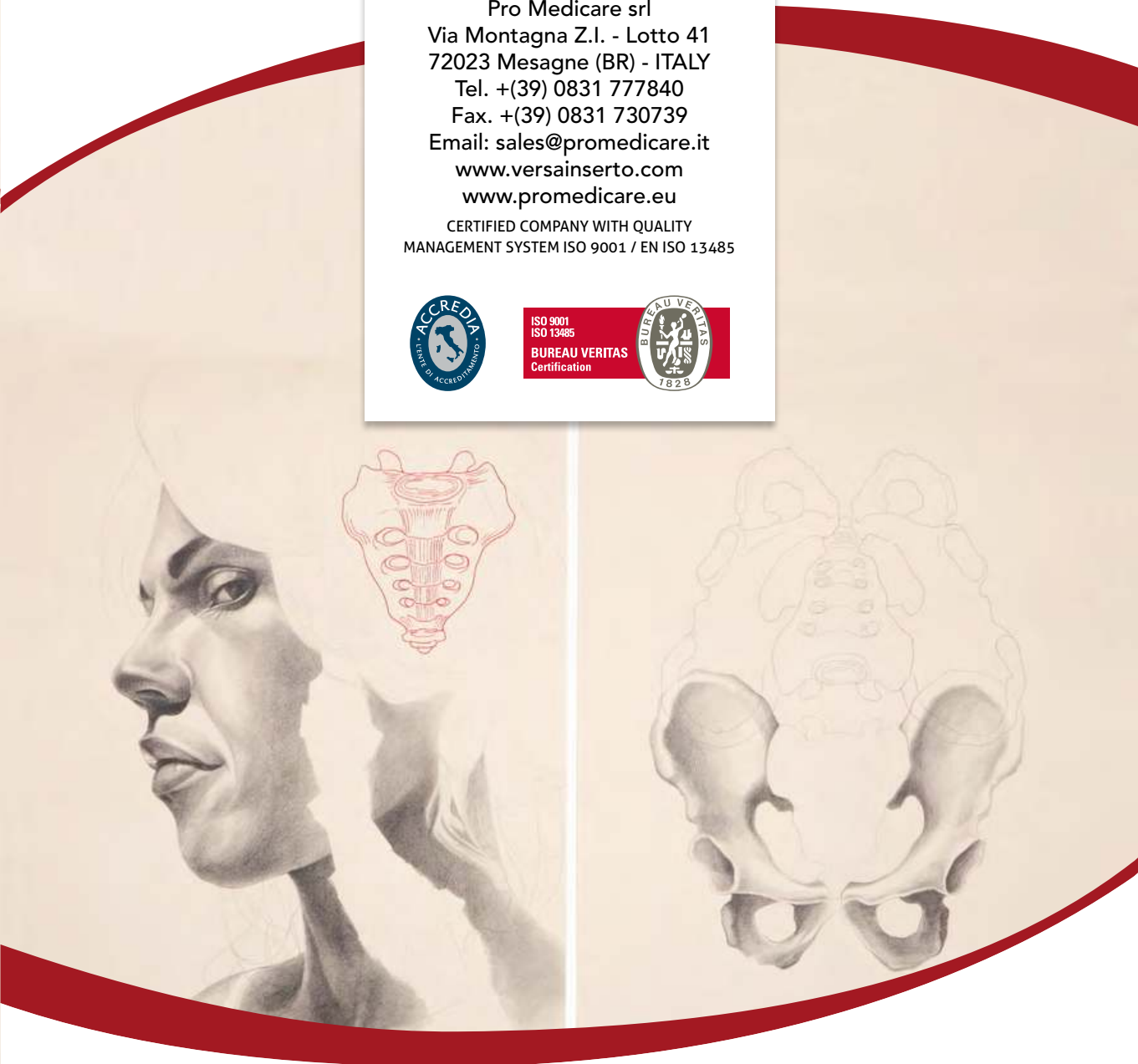

DISTRIBUTED BY:



CE 2 YEAR WARRANTY

www.versainserto.com